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DESCRIPTION

CombiHaler® / CombiHaler Solo® is a spacer designed for invasive and noninvasive mechanical ventilation systems. CombiHaler® allows the simultaneous use of a pressurized metered dose inhaler and a vibrating mesh nebulizer.

SPECIAL PRECAUTIONS

- CombiHaler® / CombiHaler Solo® ships fully assembled. It should never be opened before use. Any CombiHaler® / CombiHaler Solo® which has been opened should be discarded and replaced.
- When commissioning, it is necessary to ensure that:
 - The device does not contain any foreign body.
 - The caps are positioned correctly (in closed position) when no MDI and / or nebulizer is used.
 - The MDI nozzle functions properly (tested visually with a puff of product.)
- During use it is necessary to ensure that:
 - The caps are positioned correctly when no MDI and / or nebulizer is used.
 - The MDI nozzle functions properly (visual inspection during each puff of product release).

DURATION OF USE

- CombiHaler® / CombiHaler Solo® is a non-sterile device for single patient use.
- CombiHaler® / CombiHaler Solo® can be used throughout the life of the ventilating circuit for no more than seven consecutive days.
- CombiHaler® / CombiHaler Solo® should never be used on more than one circuit and must be destroyed under the same conditions as the circuit on which it was mounted.
- Any total or partial replacement of the circuit should result in the replacement of CombiHaler® / CombiHaler Solo®.

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Artikel-Nr. 381

CombiHaler® for children and adults with an opening for nebulizer

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Artikel-Nr. 382

CombiHaler®-solo for children and adults without an opening for nebulizer

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Please read through the usage directions carefully before the first use.

Die CombiHaler® / CombiHaler Solo® Verpackung beinhaltet | This CombiHaler® / CombiHaler Solo® package contains:

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The trademark CombiHaler® / CombiHaler Solo® belongs to Laboratoire ProtecSom SAS.
The patents pending belong to Tours University, INSERM and Laboratoire ProtecSom together.

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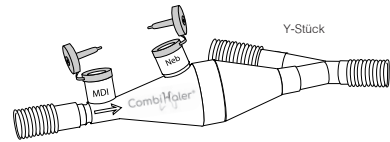
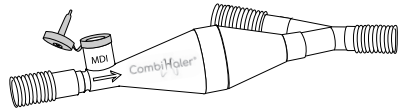
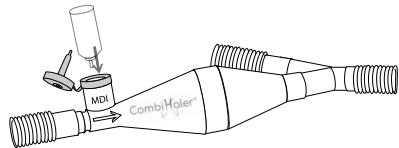
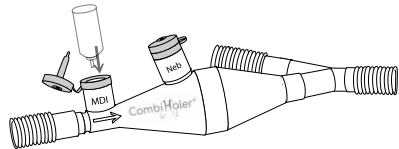
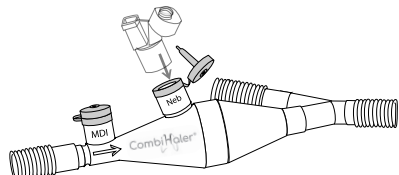
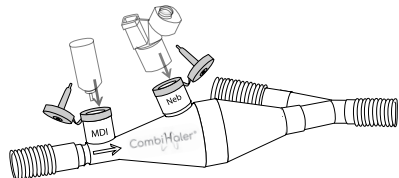
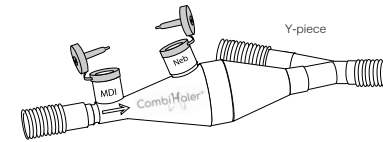
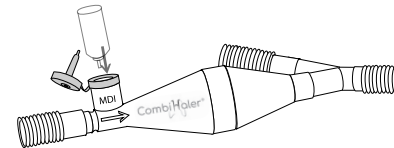
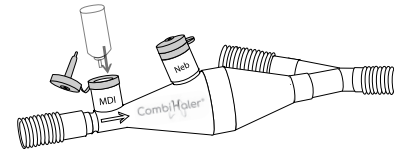
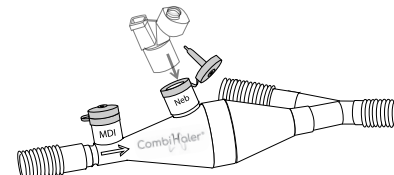
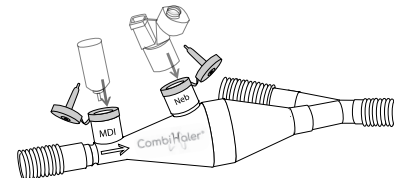
DESCRIPTION

I Mounting the circuit and use:

Insert the CombiHaler® / CombiHaler Solo® into the circuit of mechanical ventilation between the inspiratory limb and the Y-piece, following the direction of the arrow as indicated on the device CombiHaler®/ CombiHaler Solo®.

The MDI canister attaches to the port annotated MDI. For proper operation of the MDI, refer to the manufacturer indications.

Combi Haler®: The end of the nebulizer screen attaches to the orifice annotated Neb. For proper operation of the mesh nebulizer, refer to the manufacturer indications.

II Using an MDI with CombiHaler® / CombiHaler Solo®

- Shake the MDI canister according to the manufacturer's instructions.
- Open the sealing cap of the MDI port and insert the canister, valve stem facing down.
- Actuate the MDI at the beginning of the respiratory cycle.
- Let the inspiration from the ventilator administer the drug dose to the patient. Remove the canister of the MDI and replace the cap orifice in place.
- Repeat step 3 under this section II according to the number of doses needed.

III Using a vibrating mesh nebulizer with CombiHaler®

- Insert the nebulizer into the Neb port.
- Place the drug solution in the reservoir of the nebulizer.

IV For proper operation of the nebulizer, follow the instructions provided by the manufacturer.

V CombiHaler® can be used simultaneously with both an MDI and a vibrating mesh nebulizer.