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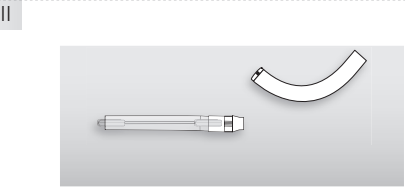
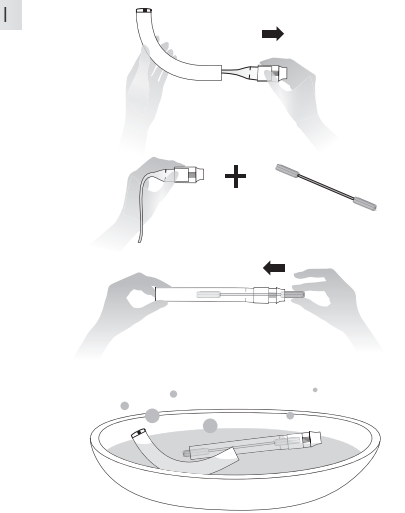
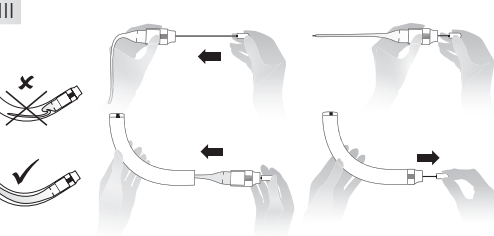
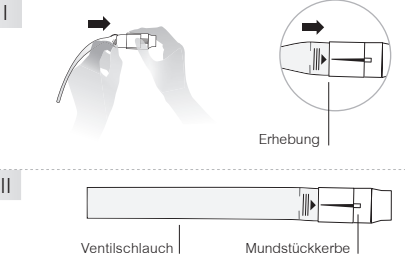
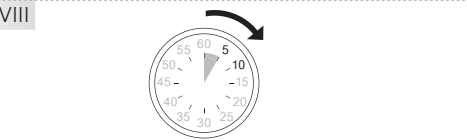
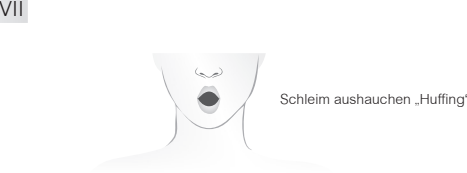
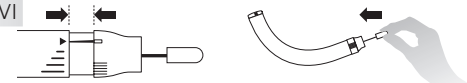
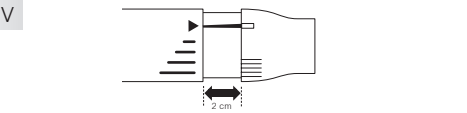
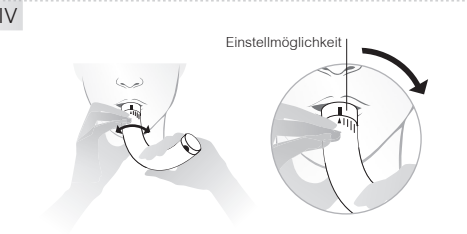
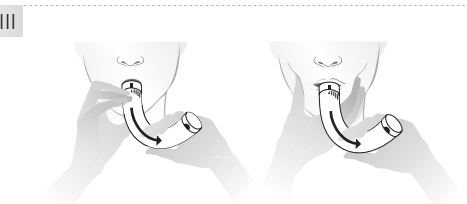
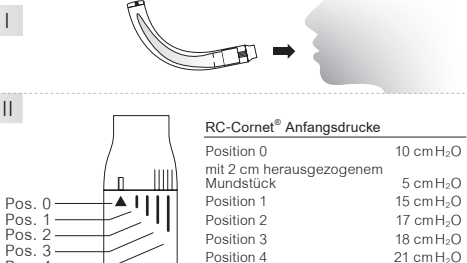
I Reinigung / Desinfektion

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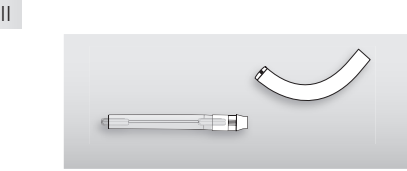
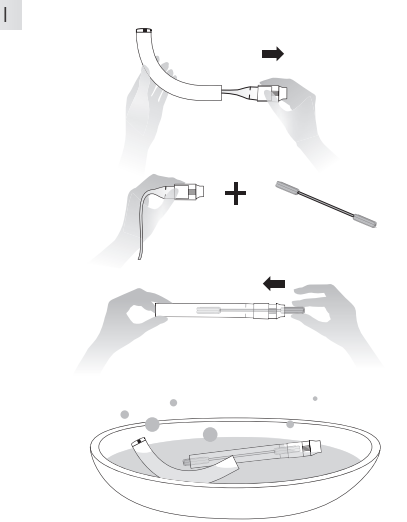
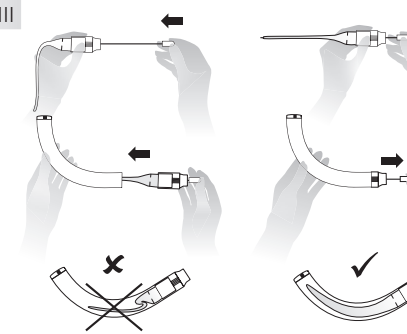
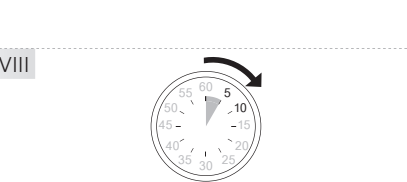
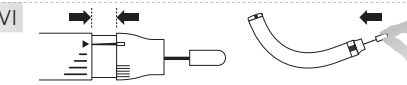
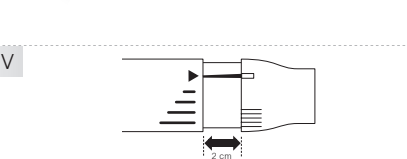
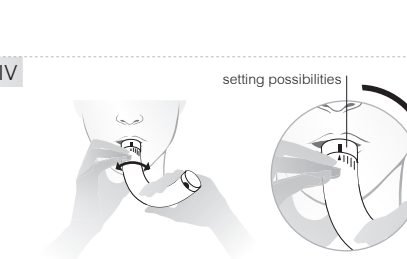
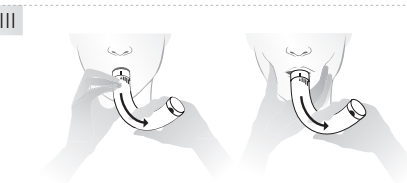
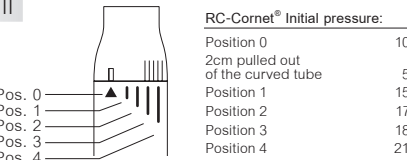
INSTRUCTIONS Cleaning

I Cleaning / Disinfection

For cleaning disconnect the mouthpiece with the valve hose from the curved tube and insert the drying spatula through the mouthpiece into the hose. Important: Throughout the cleaning process, the drying spatula must keep the valve hose open to prevent it from sticking together. Wash the parts in a solution of lukewarm clean water and standard liquid soap detergent. Rinse with clear water. After this cleaning, all parts may be cleaned in a vaporiser or for approx. 5 minutes in boiling water. Maximum temperature 134°C. If used regularly, clean the device at least twice weekly.

Further cleaning options with the RC-Clean® cleaning bag (see accessories).

TIP: For cystic fibrosis patients, we recommend remove the valve tube from the mouthpiece for cleaning.

II Drying

During the drying process, the drying spatula is designed to hold the valve tube open to prevent the tube from sticking. Shake off any excess water and let all parts dry completely. Remove the drying spatula while turning it slightly.

ASSEMBLY DIRECTIONS

I If you have removed the valve tube from the mouthpiece, attach it to the mouthpiece again, evenly up to the four elongated elevations. For insertion of the valve tube to the mouthpiece, use the insertion aid in the curved tube

II Please be sure that the flat side and the grooves align with the arrow indicator on valve tube.

III The tube cannot have kinks or bulges in the curve. For this reason, you should always use the insertion aid to guide the valve tube (and mouthpiece) back into the curved tube.

Important:
Do not forget to remove the insertion aid!
Keep all parts out of the reach of children.

INSTRUCTIONS FOR USE

I Insert the RC-Cornet® in the mouth and close the lips. Please do not bite the mouthpiece.

II Choose the mouthpiece setting for the type of treatment. To widen and stabilizing the airways, select a position between 0 to 2. Select positions 3 and 4 for mobilizing the bronchial mucus. Always start therapy at position 0 and wait 2 minutes before switching to your preferred position.

III Breath out through the RC-Cornet®. This creates positive pressure in the bronchial system as well as vibrations and a muffled sound. As you become used to breathing out through the RC-Cornet®, take deeper breaths and exhale all of the air out through the RC-Cornet®. Recommendation: Use your free hand to press your cheeks together to avoid a loss in pressure.

IV By turning the mouthpiece, you can find the most comfortable and effective setting for exact pressure and flow fluctuations. Put the RC-Cornet® in the mouth as described above and securely hold the mouthpiece with one hand. With the other hand, slowly turn the tube clockwise while blowing into it. Through blowing and slowly turning, the required pressure builds up and the resulting sound changes.

V If it is difficult to breathe out using the standard position, the mouthpiece may be pulled 2 cm out of the curved tube. The hose will then begin to vibrate at pressure as low as 5 cm of H₂O.

VI When you push the mouthpiece back into the tube, always use the insertion aid.

VII If you feel like you have to cough while blowing into RC-Cornet®, try to continue exhaling as long as you can. If you must cough, try and cough up the mucus. Imagine you want to 'fog up' a pane of glass or a mirror. By exhaling in this way, by 'huffing' as the medical term is called, it should be easier for you to expel mucus from the bronchial airways. Please do not expel phlegm into the device!

VIII The recommended usage is 3 times daily for 2-5 minutes as well as when you notice mucus obstruction. As with all types of physical therapy, the RC-Cornet® is most effective with regular usage.

ACCESSORIES AND ADDITIONAL PRODUCTS

RC-Cornet® Mouth-Nose Set
Allows the conversion of the RC Cornet® in RC-Cornet® N and replacement when worn or loss.
Article-no. 2140 | PPN 111 717 089 319

RC-Cornet® Silicone valve hose
2 pcs.
Article-no. 216 | PPN 110 045 814 776

RC-Clean®
Cleaning bag for the microwave
Easy and quick cleaning of medical products as well as baby items.
Use only for microwave safe products.
Article-no. 5000 | PPN 111 075 103 265

RC-Cornet® tracheal mask
Article-no. 3012 | PPN 111 118 816 586

Replacement parts are available online, as well as in pharmacies and medical supply stores.

RC-Cornet® is meant for use by a single patient. We recommend the exchange of the RC-Cornet® after one year of using. For clinical reuse instructions for multiple patients, visit www.cegla.de
For more information: www.cegla.de

Materials: Silicone, PP (Polypropylen)
Article no. 210 | PPN 110 841 866 756

Please report all serious incidents that have occurred in connection with the product to the manufacturer and the responsible national authority:
Landesamt für Soziales, Jugend und Versorgung
Dienststelle Mainz, Schießgartenstraße 6, 55116 Mainz
T: +49-6131-967392, F: +49-6131-96712392

CONSULT INSTRUCTIONS

DO NOT USE IF DAMAGED

NO DEHP

LATEX FREE

QUANTITY: 1

NON STERILE

MD

REF

CE

MADE IN GERMANY

CEGLA MEDIZINTECHNIK

Manufacturer and sales:
CEGLA
Medizintechnik GmbH
Horresser Berg 1
Germany
56410 Montabaur
Tel +49 2602 92 13-0
info@cegla.de
www.cegla.de

PURPOSE FOR USE

The RC-Cornet® is an aid to loosen and remove bronchial secretions associated with: Chronic obstructive pulmonary disease (COPD), bronchiectasis, mucoviscidosis (cystic fibrosis), pulmonary emphysema, unproductive cough, diminished retraction force of the pulmonary tissue, and smoker's cough.

Possible effects:

- rids irritating phlegm
- prevents unproductive coughing
- broadens airways to the pulmonary alveoli
- reduces shortness of breath
- improves gas exchange
- increases lung capacity
- lowers airway resistance
- prevents the number of stays in the hospital and the need for antibiotics

Notes:
Always begin therapy with position 0 and change to the desired position after 2 minutes. Patients with COPD, pulmonary emphysema, and smoker's cough prefer positions 0 and 2. Patients with bronchiectasis, mucoviscidosis (CF), and much viscous, firm phlegm prefer positions 3 to 4.

ADVICE: The use of the RC-Cornet® does not rely on gravity. The device may be used when lying, sitting, and standing. After you become accustomed to the RC-Cornet®, try to improve your breathing technique. Breathe in through your nose and exhale slowly through the RC-Cornet®. Important: Please expel all of your breath through the RC-Cornet®.

Physiological effects:
The RC-Cornet® works through vibrations in the chest which loosen and fluidize mucous in the bronchial tubes so that it is easier to expel.

Negative effects:
No negative effects are known from treatment with the RC-Cornet®. Hyperventilation (rapid breathing) may cause slight dizziness, or numbness in the mouth or hands. These occurrences are not dangerous and will disappear when normal breathing resumes.

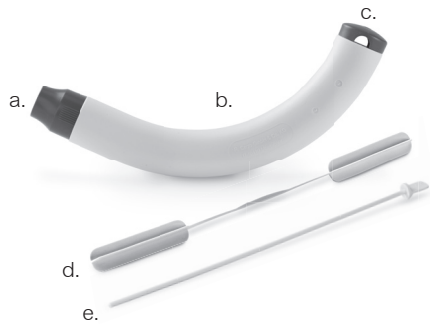
Contraindications:

- Presence of pneumothorax, blood-expectorating cough (massive haemoptysis), and cranial pressure symptoms > 20 mmHg.
- Decompensated cardiovascular diseases

RC-Cornet®

For physiotherapy of respiratory and pulmonary disease

Please carefully read through all of the usage directions before the first use.

Please clean before the first use.

This RC-Cornet® package contains:

- a. Mouthpiece with valve tube
- b. Curved tube
- c. Silencer
- d. Drying spatula
- e. Insertion aid

- Acute thromboses
- Elevated coagulation factor
- Acute blood-expectorating cough (haemoptysis)
- Possible eardrum injuries
- Pulmonary barotrauma
- Significant cardiovascular insufficiency
- Massive restriction of the respiratory pump
- Skin emphysema
- Severe infection with fever

The following relative contraindications should be discussed with your treating healthcare professional:

- Tracheoesophageal fistula
- Bronchial asthma
- Pulmonary emphysema
- Sinus infection
- Repeated, substantial nosebleeds
- Granuloma - growths in the mucous membranes near the tracheostoma
- after Surgery or injuries to the mouth, face, skull, or oesophagus

Note: Therapy following an eye operation (cataracts or glaucoma) should only start beginning the 4th week post-surgery.

NOTES: Small, white condensation deposits on the tube or in the bent tube may appear from medical talc used during production of the tubes and these deposits are harmless. The valve tube of the RC-Cornet® consists of silicone. It is recommended that the tubes are replaced every 6 months.