ANWENDUNG OSZILLIERENDE AUSATEMTHERAPIE UND EINATEMTRAINING ÜBER DEN MUND
USE OF OSCILLATING EXHALATION THERAPY AND INHALATION TRAINING THROUGH THE MOUTH

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If inhalation training is too intense and results in the patient becoming winded, or experiencing vertigo or an excessively high pulse, please stop training. The next training unit should be restarted with a lower intensity level, or with a smaller number of breaths or shorter breath lengths.

NOTE: Please abstain from any food intake shortly before and during the training/therapy. Action to be taken in the event of a severe coughing fit: Following intake of your medication, try to work slowly with the respective RC-FIT® DUO through the mouth in its initial setting, in order to also dilate the bronchi mechanically.

PURPOSE OF USE

The RC-FIT® DUO through the mouth is intended for secretion mobilisation in the case of acute and chronic diseases of the lower respiratory tract. Furthermore, the device has several applications related to breathing and its psychophysiological impact, implying a healthier life and an increased well-being. At the core is the promotion of mindfulness and stress reduction, the alleviation of pain or pain perception support for mild psychological conditions such as depression and anxiety, and, as a result, a possible improvement in sleep. The RC-FIT® DUO through the mouth is a respiratory trainer and respiratory therapy device in one and help in an emergency.

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Storage
Only store the RC-FIT® DUO through the mouth in dry condition and with closed lid. In this manner, the assistive device is protected from sunlight and dust, among other things.

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SCOPE OF DELIVERY (MATERIAL ZUSAMMENSETZUNG) RC-FIT® THROUGH THE NOSE

Nosepiece	NOSEPIECE BLUE with different intensity levels (polypropylene)
Inlay with integrated valves	s. RC-FIT® DUO through the mouth*
Intensity regulator	s. RC-FIT® DUO through the mouth*
Lid / Handle	s. RC-FIT® DUO through the mouth*

* Only included once in the scope of delivery

Δ CONTRAINDICATIONS

Training and therapy with the RC-FIT® DUO through the nose produce in a negative pressure during inhalation, and an overpressure during exhalation. Therefore, contraindications for its use are:

- Presence of pneumothorax, blood-expectorating cough (massive haemoptysis), and cranial pressure symptoms >20 mmHg.
- Decompensated cardiovascular diseases
- Acute thromboses
- Elevated coagulation factor
- Acute blood-expectorating cough (haemoptysis)
- Possible eardrum injuries
- Pulmonary barotrauma
- Significant cardiovascular insufficiency
- Massive restriction of the respiratory pump
- Skin emphysema
- During a purulent nasal discharge
- Severe infection with fever

The following relative contraindications should be discussed with your treating healthcare professional:

- Tracheoesophageal fistula
- Bronchial asthma
- Pulmonary emphysema
- Sinus infection
- Repeated, substantial nosebleeds
- Granulo ma - growths in the mucous membranes near the tracheostoma
- In case of acute sinusitis or middle ear infection,
- After surgery or injuries to the mouth, face, skull, or oesophagus

Note: The timely use after a nose operation should be discussed in advance with the attending doctor. Training / therapy following an eye operation (cataracts or glaucoma) should only start beginning the 4th week post-surgery.

SIDE EFFECTS

At the beginning of the treatment, you may experience a runny nose for a short while. This is a regulatory phenomenon. If you get out of breath during training or experience dizziness or a significantly increased pulse, please stop training. Please restart the next training session at an easier intensity level or with a lower number or length of breaths.

Other possible side effects

- Elevated intracranial pressure
- Cardiovascular risk > Insufficient oxygen supply to the heart > Decreased venous return

NOTE: Due to the function of alternate nostril breathing, the sounds may vary or be delayed. Regular use of RC-FIT® DUO through the nose can improve this. In cases of pronounced polyd formations and/or a deviated septum, use may only be possible on one side and/or the course of therapy may be unpredictable.

INTENDED USE

The RC-FIT® DUO through the nose is suitable for the treatment of both acute and chronic respiratory conditions. Furthermore, the device has several applications related to breathing and its psychophysiological impact, implying a healthier life and an increased well-being. At the core is the promotion of mindfulness and stress reduction, the alleviation of pain or pain perception, support for mild psychological conditions such as depression and anxiety, and, as a result, a possible improvement in sleep. The RC-FIT® DUO through the nose is a respiratory trainer, relaxation device, and respiratory therapy device all in one.

Duration of use
If used daily, we recommend the exchange of the RC-FIT® DUO after 1 year. Please notify the manufacturer of all serious incidents occurring in connection with the device.

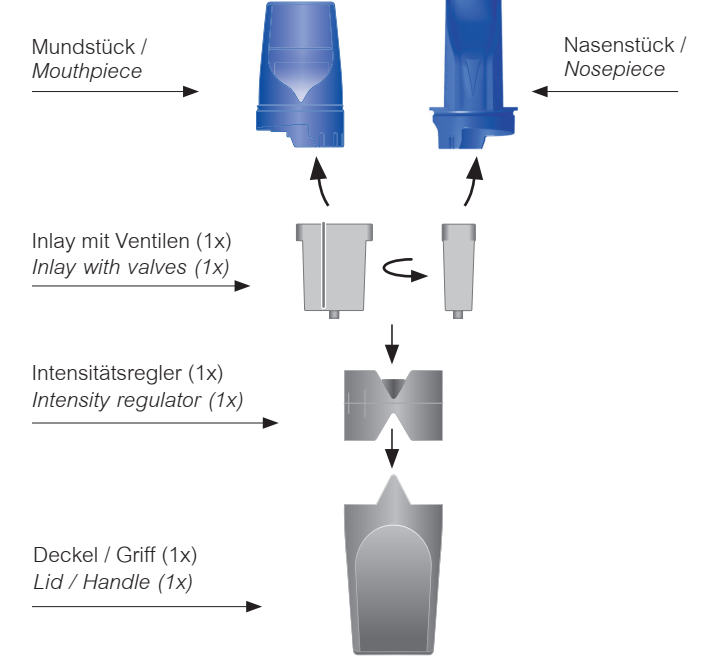
Storage
Store the RC-FIT® DUO through the nose in a dry place in the accompanying outer packaging and protect the device from sunlight and dust, among other things.

Disposal
The RC-FIT® DUO can be disposed of with household waste, as long as not prescribed otherwise by the respective country's disposal regulations.

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Δ WARNING

Please clean before the first use! Please ensure that the RC-FIT® DUO is used in accordance with these Instructions for Use. Its incorrect use (off-label use) will result in an exclusion of liability for the manufacturer.

IMPORTANT INFORMATION

Before using the RC-FIT® DUO for the first time, please read the Instructions for Use carefully. If you have additional questions regarding its usage, please consult with your qualified medical practitioner. The RC-FIT® DUO is well-suited for users starting at 3 years of age in home and clinical setting. The patient must be able to purposely coordinate inhalation and exhalation through the mouth. Children should only undergo training/therapy with the assistance of an adult. In the case of severely impaired patients, the training/therapy should be conducted together with a nurse or physiotherapist. Paraplegic patients or persons who suffer from neuromuscular diseases should only undergo training/therapy on the advice of qualified medical practitioners. After taking your medication, slowly work with the RC-FIT® DUO in the starting position to mechanically widen your airways. If the sound is constant, then the RC-FIT® DUO is being used correctly, and the necessary flow and pressure fluctuations are being achieved. Important: All individual parts must be stored out of the reach of children. After every use, please allow the respective RC-FIT® DUO to dry as moisture may form in the product with long-term training/therapy.

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RC-FIT® DUO THROUGH THE MOUTH

A) TRAINING & THERAPY WITH VIBRATIONS

As it is in sports and everyday life, having more oxygens means having a greater ability to function. The inlay with integrated valve produces oscillations that are acoustically perceived as sounds. These expand in the airways. As a result, shortness of breath is decreased during physical exertion. Mucilage and coughing in acute and chronically ill patients can be reduced. Through stabilisation of the airways more oxygen can be taken in. In addition, body function perception is achieved. The goal is to support the cleaning function of the airways in the best possible manner. This can be very effective within the scope of simple colds all they way to chronic pulmonary diseases. Aside from moistening the airways with fresh, moist air (if there is no fever present), inhalations, or moistening nasal sprays in order to keep the cleaning mechanism working, the PEP therapy device can also free the airways of stubborn mucus. The breeding ground for infections is thus minimised.

B) ANCHOR FOR SHORTNESS OF BREATH

As with the pursed lips technique, the RC-FIT® DUO through the mouth can stabilise the bronchi in cases of shortness of breath, through a deliberately slow exhalation. Hyperventilation can be reduced or prevented. Thanks to its practical pocket shape, the device is always close at hand in the lowest intensity level. Practice removing with the RC-FIT® DUO through the mouth and the use pursed lip technique about 60 times to be optimal for emergencies to be armed.

USE

Use of the RC-FIT® DUO through the mouth is not dependent on the force of gravity. The device may thus be used in a reclining, sitting or standing position. For the best effect, however, using the device while standing is preferable. Drink a glass of water before training in order to keep your airways moist.

I. Please inspect the device for foreign bodies or damages prior to every use. If any parts are missing or damaged, replace them immediately.

II. Start your first training session in the lowest intensity level "level I" (largest-possible triangle). The smaller the triangle becomes, the higher your training and therapy load (intensity). RC-FIT® DUO through the mouth has a total of 4 intensity levels.

Average pressure during exhalation / inhalation

intensity level	Flow rate (pressure difference to ambient pressure)			
	5/-5 l/min	10/-10 l/min	15/-15 l/min	20/-20 l/min
I	0,5/-0,6 cmH ₂ O	1,7/-1,4 cmH ₂ O	3,3/-3,2 cmH ₂ O	5,4/-4,2 cmH ₂ O
II	0,5/-0,5 cmH ₂ O	1,9/-1,7 cmH ₂ O	3,9/-3,7 cmH ₂ O	6,6/-5,3 cmH ₂ O
III	0,6/-0,6 cmH ₂ O	2,2/-2,2 cmH ₂ O	4,9/-4,8 cmH ₂ O	8,4/-8,3 cmH ₂ O
IV	0,7/-0,7 cmH ₂ O	2,2/-2,3 cmH ₂ O	5/-5,3 cmH ₂ O	8,4/-8,7 cmH ₂ O

Frequency during inspiration: 165 Hz +/-5 %
Frequency during exhalation: 130 Hz +/-5 %

III. A) Place the RC-FIT® DUO through the mouth in your mouth and close your lips around the mouthpiece. Please do not close the triangle with your fingers.

Breathe in and out through the mouth.

If the sound at the start of each breathing manoeuvre is loud and then quickly becomes quieter, please adapt your breathing accordingly.

Make sure that the sound when breathing through the RC-FIT® DUO through the mouth is uniform and continuous. In such a case you will know that you have surrounded the training load. Concentrate on the sounds occurring with your breathing. Try to produce these sounds to be as balanced as possible, for as long as possible.

Depending on your training condition, take a 1-minute break after 1½ minutes. One training unit should consist of 10–30 breaths, 3 x daily. In the case of Parkinson's with dysphagia, training of 5 x daily with 5 exhalations each time should be conducted.

B) To train for cases of shortness of breath or while experiencing shortness of breath always use only the "I" intensity level. Try to breathe in the goalkeeper position or the cart driver position calmly and in a prolonged manner, via the RC-FIT® DUO through the mouth. Repeat this breathing cycle approx. 10 times if possible.

If the training is too light, or the sound is too loud even after 30 breaths, increase the intensity (resistance) by rotating the mouthpiece, see Fig. II.

CLEANING AND DISINFECTION (Boiling)

If used regularly, clean and disinfect the device once a week, please clean daily if there is a lot of mucus. If necessary (e.g. if no more sounds are produced), insert parts of the RC-FIT® DUO through the mouth in a warm, light detergent solution and then boil out.

- Cleaning and disinfection**
For cleaning purposes, pull off the mouthpiece from the intensity regulator. Then, carefully remove the inlay with integrated valves from the intensity regulator. **IMPORTANT:** Try to avoid any contact with the integrated valves! Wash all parts in a solution of lukewarm clean water and standard liquid soap detergent. Rinse with clear water. After this cleaning, disinfect all parts in a vaporisator or in boiling water for approx. 5 minutes.
- Drying**
In order to prevent mould formation, all parts must be completely dried. To accelerate this process, shake off any excess water before setting the parts down. Allow all parts to air dry. **Do not rub dry the inlay with towels.**
- Assembly**
Before assembly, please wash your hands and ensure that all parts of the RC-FIT® DUO are clean. There may be no dust residue or scattered particle deposits, as these could be breathed in and reach the lungs.

FREQUENCY OF USE

Recommended therapy intervals for the respiration:
RC-FIT® DUO: 3 x daily 10–30 breaths, 7 days a week. Additional usage when you notice heavy mucus obstruction or occasional shortness of breath.

TIPS:

- Integrate the therapy and training into your daily routine, especially when you notice very little mucus, e.g. always before brushing your teeth.
- To obtain a better effect from your medication spray, integrate this inhalation into therapy with the RC-FIT® DUO.

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RC-FIT® DUO THROUGH THE NOSE

POSSIBLE EFFECTS

- Reactivates natural breathing
- Improves ventilation of the sinuses and middle ear
- Reduces shortness of breath, caused by e.g. psychological factors
- Reduces the breathing rate
- Strengthens the respiratory muscles
- Improves gas exchange
- Freeds the upper airways from irritating phlegm
- Reduces coughing due to post-nasal-drip
- Reduces bronchial hyperreactivity at asthma
- Enhances drug deposition

USE

Use of the RC-FIT® DUO through the nose is not dependent on the force of gravity. The device may thus be used in a reclining, sitting and standing position. According to the guidelines*, we recommend cleaning your nose with a nasal rinse before using the RC-FIT® DUO through the nose if you have sinusitis. Alternatively, you can use a moisturizing nasal spray (e.g., RC-Pflege® N). Experience has shown that this results in a faster and more pronounced therapeutic effect.

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II. Always start your first training session in the lowest intensity level "MINIMUM" (largest-possible triangle).

The smaller the triangle, the higher your training and therapy load (intensity). The intensity levels on the nosepiece are continuously adjustable.

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III. Please blow your nose before beginning the training. Hold the RC-FIT® DUO through the nose as shown. Insert the round opening of the nosepiece of the RC-FIT® DUO through the nose into one nostril and close the other nostril with the index finger of your other hand. Ensure that the triangle of the intensity control remains open at all times.

Breathe in through your nose and after a short break, breathe out through your nose again.

Repeat the process with the other nostril after completing the training session (approx. 10–30 breaths). If the sound at the start of each breathing manoeuvre is loud and then quickly becomes quieter, please adapt your breathing accordingly. Make sure that the sound is uniform and continuous when breathing through the RC-FIT® DUO through the nose. By this, you will know that you have reached the intended training load. Concentrate on the sounds created by your breathing. Try to produce these sounds as balanced as possible, for as long as possible. The sounds may be slightly delayed depending on the intensity. If no sound is heard, try adjusting the angle slightly and start the breathing very softly. Depending on your training condition, take a 1-minute break after 1½ minutes. One training unit should consist of 10–30 breaths, 3 x daily, per nostril.

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If used regularly, clean and disinfect the device once a week, please clean daily if there is a lot of mucus. If necessary (e.g. if no more sounds are produced), insert parts of the RC-FIT® DUO through the nose in a warm, light detergent solution and then boil out.

- Cleaning and disinfection**
For cleaning purposes, pull off the nosepiece from the intensity regulator. Then, carefully remove the inlay with integrated valve from the intensity regulator. **IMPORTANT:** Try to avoid any contact with the integrated valve! Wash all parts in a solution of lukewarm clean water and standard liquid soap detergent. Rinse with clear water. After this cleaning, disinfect all parts in a vaporisator or in boiling water for approx. 5 minutes.
- Drying**
In order to prevent mould formation, all parts must be completely dried. To accelerate this process, shake off any excess water before setting the parts down. Allow all parts to air dry. **Do not rub dry the inlay with towels.**
- Assembly**
Before assembly, please wash your hands and ensure that all parts of the RC-FIT® DUO are clean. There may be no dust residue or scattered particle deposits, as these could be breathed in and reach the lungs.

FREQUENCY OF USE

Recommended therapy intervals for the respiration:
RC-FIT® DUO: 3 x daily 10–30 breaths, 7 days a week. Additional applications whenever needed.

TIPS:

- Integrate the therapy and training into your daily routine, especially when you notice very little mucus, e.g. always before brushing your teeth.

*Guideline of the German Otorhinolaryngology Society, Head and Neck Surgery and the German Society of General and Family Medicine, DEGAM Guideline (Sinusitis)