



# A Clear Nose, without Medicine

The RC-Cornet® N (Nasal Cornet) clears the nose in a natural way. It has a long-term decongestant effect, is mucolytic and regenerative.



**RC**  
MEDICAL TECHNOLOGY



Therapy with the RC-Cornet® N (Nasal Cornet) clears the nose and sinuses in a natural way.

The Nasal Cornet belongs to the oscillating PEP systems which by means of a positive exhale pressure of the respiratory tracts offer treatment on a physical basis.


The positive pressure, which develops in the nose cornet while exhaling, is transmitted over the entire nasopharyngeal zone. The mucus membrane is thereby physically massaged and the upper respiratory tracts are ventilated – without any use of medication.

Through the soft pressure massage of the RC-Cornet® N Therapy the upper respiratory tracts are cleared of mucus, the lymph flow is stimulated and the openings of the sinuses are noticeably expanded and ventilated.

The RC-Cornet® N can be administered standing up, sitting or lying down and is suitable for adults and children.

## RC-Cornet® N Scope of Effectiveness

The scope of the RC-Cornet® N ranges from the entire area of the sinuses to the pharynx.

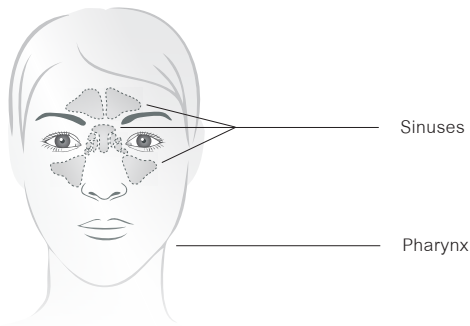

## Advantages of the RC-Cornet® N Therapy


- Clears and cleanses the upper respiratory tracts from mucus
- Alleviates coughing from postnasal-drip-syndrome
- Decreases the bronchial hyperactivity\*
- Helps with nose spray addiction (rhinitis)
- Improves the ventilation of the sinuses and the middle ear
- Optimizes the dispersion of medicine

\*Studies available upon request


The ailments of chronic sinusitis, postnasal-drip-syndrome, disorders of the tuba auditiva (ventilation disorders of the Eustachian tube), bronchial hyperactivity and nose spray addiction (rhinitis) are alleviated with regular use.

These include, among others, a stuffed up nose, headaches, fatigue and exhaustion, swollen eyes and a limited sense of smell. From the very first use of the RC-Cornet® N the therapy is noticeable.

The effectiveness of the RC-Cornet® N Therapy can clearly be seen on a thermo monitor.

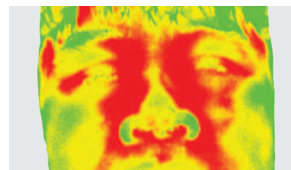

Initial findings of a chronic sinusitis with clearly swollen mucous membranes.

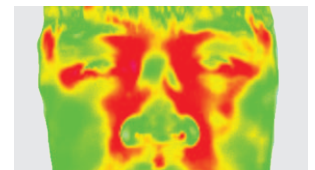

After using the RC-Cornet® N a definite reduction in the swelling of the mucous membranes resulting in the opening of the paranasal sinuses. The nose is freer.

\*Source: Cegla U.H., Jost H.-J., Harten A., Changes in Postnasal-Drip, dry cough, respiratory congestion and bronchial hyperactivity with bronchial asthma through oscillating PEP-Treatment (RC-Cornet® N) of the nose-throat area: Dustri publishing house Munich; Respiratory, Lung hospital, year 29, no. 9/2003, S. 428-436

## RC-Pflege® N supports the RC-Cornet® N Therapy

The effectiveness of the RC-Cornet® N Therapy can be increased when, before the application, physiologic salt solution is inserted into the nose.

The RC-Pflege® N is perfectly suited for this. The nose and pharynx spray cleanses, protects and cares for the mucus membrane. It moistens the nose and pharynx mucus membrane and supports the quick regeneration as well as the wound healing of the nose mucus membrane. In addition it is ideally suited for patients of long-term oxygen therapy.

On the contrary to most nose sprays with decongestive effects the composition of the RC-Pflege® N allows for a daily and lasting application, without the danger of becoming addicted (privinismus).

# RC PFLEGE<sup>N</sup>

## Nose and Pharynx Spray

Protects and cares  
for nose and pharynx



[www.cegla-shop.com](http://www.cegla-shop.com)


## Effective, simple and good for

- Chronic sinusitis
- Chronic mucus production in the nasopharyngeal zone
- Asthma with postnasal-drip-syndrome
- Ventilation disorders of the sinuses and middle ears
- Sin bronchial syndrome
- Nose spray addiction (privinismus)

MORE EFFECTIVE:

## Combination Therapy

The combination of the RC-Cornet® N with an inhalation therapy via a nebulizer (humidity inhaler) improves medication deposition, reduces the time of therapy delivery and increases the effectiveness of both therapies.

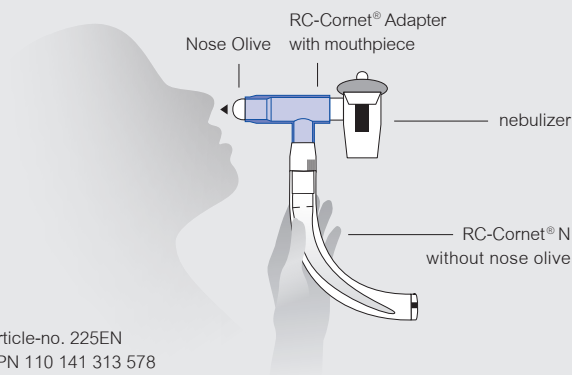

Article-no. 225EN  
PPN 110 141 313 578  
[www.cegla-ltd.com](http://www.cegla-ltd.com)

The RC-Cornet® Adapter with mouthpiece combines the RC-Cornet® N with most commonly used nebulizers. Pull the nose olive out of the RC-Cornet® N. Place this in the RC-Cornet® Adapter with mouthpiece and then onto the converted adapter on the RC-Cornet® N. Now connect your nebulizer with the RC-Cornet® Adapter.



## Product Details

RC-Cornet® N

Article-no. 220EN

PPN 110 886 232 700 | PIP 393-4809



## Accessories for the Combined Therapy

RC-Cornet® adapter with mouthpiece

Article-no. 225EN | PPN 110 141 313 578



## Additional Products

RC-Cornet® N valve hoses natural rubbers

Twin pack

Article-no. 211EN | PPN 110 875 621 700



RC-Clean

Cleaning bag for the microwave

Easy and quick cleaning of medical products as well as baby items. Use only for microwave safe products. No metal can be used with the product and it is not working in an oven.

Article-no. 500EN

PPN 111 075 103 265 | PIP 393-6424



Sales Enquiries:

R.Cegla Ltd.  
PO Box 366  
Mellis  
EYE  
IP23 9AH  
Mb +44 7944 623237  
contact@cegla-ltd.com  
www.cegla-ltd.com

Further information:  
[www.cegla-ltd.com](http://www.cegla-ltd.com)

Available in selected  
pharmacies or at  
[www.cegla-shop.com](http://www.cegla-shop.com)