## RC-CORNET<sup>®</sup> ADAPTER WITH MOUTHPIECE

To combine the respiratory therapy and inhalation therapy with the RC-Cornet<sup>®</sup> and the RC-Cornet<sup>®</sup>N


Please read the RC-Cornet<sup>®</sup> adapter with mouthpiece instructions for use carefully before first use. Please also follow the instructions of the nebulizer (the nebulizer is not included).

**CEGLA**  
MEDICAL TECHNOLOGY

R. Cegla GmbH & Co. KG  
Horresser Berg 1  
56410 Montabaur  
Germany  
Tel +49 2602 9213-0  
info@cegla.de  
www.cegla.de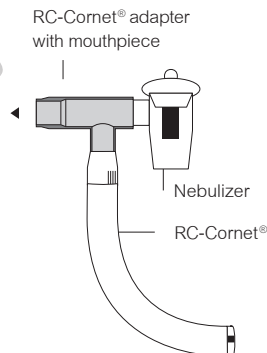

### INSTRUCTION FOR USE I

For the combination therapy of the lower respiratory tracts


The RC-Cornet<sup>®</sup> adapter combines the basic cornet with all common nebulizers.

Remove the mouthpiece of the nebulizer first. Now connect the RC-Cornet<sup>®</sup> adapter with mouthpiece to the nebulizer (see image).

Caution: The mouthpiece of the nebulizer must first be removed.

Then slide the adapter onto the RC-Cornet<sup>®</sup> (basic cornet) so that it fits tightly. Set the RC-Cornet<sup>®</sup> to the desired pressure profile by turning the mouthpiece at the bent end. Enclose the mouthpiece of the RC-Cornet<sup>®</sup> adapter with your lips, inhale and breathe in the aerosol and then exhale against the generated oscillating resistance.

**Recommended frequency of the combination therapy:** 2–3 x daily, 2–5 minutes (with severe mucous buildup more often).

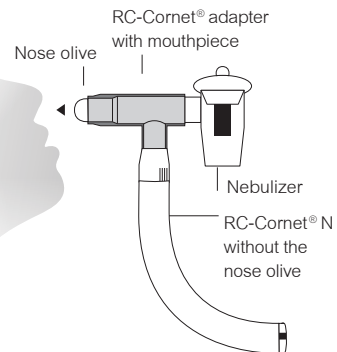### INSTRUCTION FOR USE II

For the combination therapy of the upper respiratory tracts

The RC-Cornet<sup>®</sup> adapter combines the nose cornet with all common nebulizers.

Pull the nose olive out of the RC-Cornet<sup>®</sup>N (nose cornet). Place it into the RC-Cornet<sup>®</sup> adapter and then add the converted adapter onto the cornet.

Connect the system to the nebulizer, so that it fits tightly (see image). Caution: The mouthpiece of the nebulizer must first be removed.

Set the desired pressure profile (setting positions 0–1) on the RC-Cornet<sup>®</sup> by turning the mouthpiece at the bent end.

Close off one nose hole with your pointer finger and push the nose olive of the RC-Cornet<sup>®</sup> adapter into the other nose hole.

Breathe in the aerosol through the nose olive and generate the healing pressure oscillation in the nose-throat area by exhaling.

**Recommended frequency of use of the combination therapy:** 2–3 x daily, 2–5 minutes (with severe mucous buildup more often).

## CLEANING RECOMMENDATIONS

The RC-Cornet<sup>®</sup> adapter with mouthpiece can be cleaned as follows:

- In the RC-Clean cleaning bag for the microwave (sold separately)
- Boiling
- In a vaporizer
- With hot steam: temperatures up to 134°C

Clean the RC-Cornet<sup>®</sup> adapter with mouthpiece every 2 days!

The RC-Cornet<sup>®</sup> adapter with mouthpiece is meant for use by a single patient. For clinical reuse instructions for multiple patients, visit [www.cegla-ltd.com](http://www.cegla-ltd.com)

article-no. 225EN | PPN 1101 4131 3578

Other products available in pharmacies, medical supply stores or at [www.cegla-shop.com](http://www.cegla-shop.com)

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