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RC-CORNET® PLUS / RC-CORNET® PLUS TRACHEO / RC-CORNET® PLUS NASAL

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## HYGIENIC RE-USE

RC-CORNET® PLUS / RC-CORNET® PLUS TRACHEO / RC-CORNET® PLUS NASAL

Prior to use by a new patient, RC-Cornet® PLUS / RC-Cornet® PLUS TRACHEO / RC-Cornet® PLUS NASAL must be sterilized.

### 1 CLEANING AND DESINFECTION

---

- 1.1 Take the entire RC-Cornet® PLUS / RC-Cornet® PLUS TRACHEO / RC-Cornet® PLUS NASAL apart as follows. The following order is mandatory.

Remove the body lid. / Remove the adapter from the body. / Remove the mouth-piece, the tracheal tube or the nosepiece with nozzle from the adapter. / Pull the one way valve out of the adapter. / Press the upper section of the valve tube together so that the valve tube can be pulled out of the adapter. / Insert the cleaning aid into the valve tube.

- 1.2 Place the individual parts in the instrument washer.

- 1.3 Select the 93°C program (reaction time 5 minutes). When using cleaning agents, please follow the manufacturer's instructions. All cleaning agents on the VAH List approved by the German Association of Applied Hygiene may be used for RC-Cornet® PLUS / RC-Cornet® PLUS TRACHEO / RC-Cornet® PLUS NASAL.

The following alkaline cleaning products may be used:

- Neodisher® Mediclean forte / Manufacturer:  
Dr. Weigert GmbH & Co. KG, Hamburg
- thermosept® X-tra / Manufacturer: Schülke & Mayr GmbH, Norderstedt

- 1.4 Shake to remove excess water and allow all parts to air-dry.  
Important: The cleaning aid must remain in the tube during the entire cleaning process.

### 2 STERILIZATION

---

- 2.1 Store RC-Cornet® PLUS / RC-Cornet® PLUS TRACHEO / RC-Cornet® PLUS NASAL disassembled in the sterilization package provided.

- 2.2 RC-Cornet® PLUS / RC-Cornet® PLUS TRACHEO / RC-Cornet® PLUS NASAL can be sterilized as follows: 121 °C for min. 15 minutes or 134 °C for 5 minutes.

- 2.3 Prior to re-use, follow the Instructions for Use to reassemble all RC-Cornet® PLUS / RC-Cornet® PLUS TRACHEO / RC-Cornet® PLUS NASAL parts.

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- Neodisher® Mediclean forte / Hersteller: Dr. Weigert GmbH & Co. KG, Hamburg
- thermosept® X-tra / Hersteller: Schülke & Mayr GmbH, Norderstedt