

RC-Cornet[®] nosepiece set


With the RC-Cornet[®] nosepiece set, you can easily convert the RC-Cornet[®] for sinus therapy.

The RC-Cornet[®] nosepiece set contains:


- Nosepiece with nose olive and valve hose
- Drying spatula
- Insertion aid



How to convert your RC-Cornet[®]:

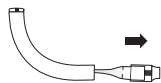

Pull the mouthpiece with the valve tube out of the RC-Cornet[®].

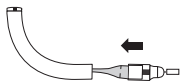

Place the RC-Cornet[®] nosepiece set into the curved tube using the insertion aid provided.

Please carefully read the instructions for use for the RC-Cornet[®] N before the first use.

ACCESSORIES AND ADDITIONAL PRODUCTS

RC-Cornet® N valve hoses natural rubber

Double pack

Article-no. 211EN
PPN 110 875 621 700



RC-Clean

Cleaning bag for the microwave

Easy and quick cleaning of medical products as well as baby items.
Use only for microwave safe products.

Article-no. 500EN
PPN 111 075 103 265 / PIP 393-6424



RC-Pflege® N

Nose and Throat Spray

Protects and cares for nose and throat.

Article-no. 250EN



Accessories and additional products are available in pharmacies, medical supply stores or at:

www.cegla-shop.com

All pieces are available seperately.

Further information: www.cegla.de

Article-no. 221EN

PPN 110 886 233 363

05/2017

CEGLA
MEDICAL TECHNOLOGY

Manufacturer:

R. Cegla GmbH & Co. KG
Horresser Berg 1
56410 Montabaur
Germany

www.cegla.de